Polycheck[®] [RAST-compatible]

Allergy Diagnostic Cassette-Format

Scanner based assay for screening and quantification of allergen specific IgE (kU/ I and RAST-classification).

REF:

20 Allergens

05012001 Food
(05012021 for automated use)

05012002 Inhalation
(05012022 for automated use)

05012003 Paediatric
(05012023 for automated use)

05012008 Insect-Epithelia

05012009 rBet v1/v2 plus

05012005 Korea I-Standard
(05012015 for automated use)

05012006 Korea II-Inhalation
(05012016 for automated use)

05012007 Korea III-Food
(05012017 for automated use)

10 Allergens

05012101 Deca-Food I
(05012111 for automated use)

05012201 Deca-Food II
(05012211 for automated use)

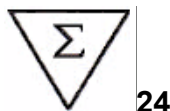
05012102 Deca-Inhalation I
(05012112 for automated use)

05012202 Deca-Inhalation II
(05012212 for automated use)

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1 = Shrimp Pacific, Atlantic; 2 = Cockroach German, Oriental, American;
3 = Ash European, black, Oregon, white; 4 = Lemon, Lime, Orange, Mandarin;

Materials Supplied, Storage and Stability

Component	Content	Preparation	Store at	Shelf Life
Polycheck® Food: red	24 Cass.	ready to use	2 - 8 °C with desiccant in a carefully sealed plastic bag	until the expiry date
Polycheck® Inhalation: blue	24 Cass.			
Polycheck® Paediatric: turquoise	24 Cass.			
Polycheck® Deca-Food I: orange	24 Cass.			
Polycheck® Deca-Food II: purple	24 Cass.			
Polycheck® Deca-Inhalation I: light green	24 Cass.			
Polycheck® Deca-Inhalation II: dark green	24 Cass.			
Polycheck® Korea I-Standard: light green	24 Cass.			
Polycheck® Korea II-Inhalation: dark green	24 Cass.			
Polycheck® Korea III-Food: orange	24 Cass.			
Polycheck® rBet v1/v2 plus: purple	24 Cass.			
Polycheck® Insect-Epithelia: olive	24 Cass.			
Start Solution, buffered protein solution	2 x 3,5 ml, blue (1 x 18ml Automat)	ready to use	2 - 8 °C	30 days after opening, resp. until the expiry date
anti-IgE Antibody, monoclonal (murine) antibody labeled with ligand	2 x 3,5 ml green (1 x 18ml Automat)	ready to use	2 - 8 °C	30 days after opening, resp. until the expiry date
Enzyme-Labeled anti-Ligand, Ligand conjugated to alkaline phosphatase	2 x 3,5 ml white (1 x 18ml Automat)	ready to use	2 - 8 °C	30 days after opening, resp. until the expiry date
Substrate Solution, 5'bromo-4'chloro-3' indolylphosphate and 4' nitroblue tetrazolium, buffered	2 x 3,5 ml black (1 x 18ml Automat)	ready to use	2 - 8 °C protect from light	30 days after opening, resp. until the expiry date
Wash Buffer, phosphate buffer, pH 7.4	1 pouch	dissolve in 1 liter demineralized water	2 - 8 °C avoid foaming	30 days after dissolving, resp. until the expiry date

Silinmiş: marked on the label

Material Safety Data Sheets are available on request.

Materials Required

- personal computer (Windows 98, Windows ME, Windows 2000, Windows XP), printer
- flat bed scanner (ask for recommended models)
- Biocheck Imaging Software (BIS) for patient-oriented analysis, calculation and report
- rocking shaker (30rpm)
- adjustable pipettes for 200 up to 1000 µl; Multipette (1ml) to dispense the wash solution
- demineralized water
- one-liter bottle (for wash solution)

Silinmiş: diluted

Specimen Collection and Preparation

Serum samples may be stored refrigerated at 2-8 °C for up to 3 days. For longer periods samples should be stored frozen at -20°C. Lipemic sera should be cleared by centrifugation prior to assay.

Warnings and Precautions

All reagents of this test kit are strictly intended for *in vitro* diagnostic use only. Use by staff, being specially informed and trained in methods dealing with *in vitro* diagnostics. Please adhere strictly to the sequence of pipetting steps provided in this protocol.

Sample material of patients (for example serum or plasma) normally used in laboratory determinations are always classified as potentially infectious. Samples of risk patients should be specially labeled and if necessary be handled in safety work benches (for example laminar flow bench).

Polycheck®

Method and Test Principle

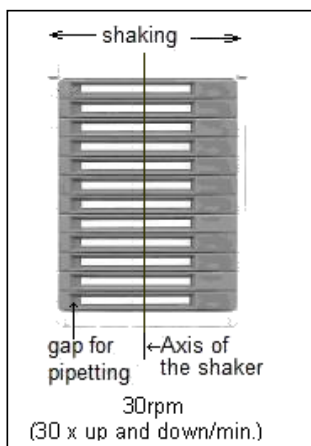
The Polycheck[®] screening assay is an enzyme immuno assay for the quantitative measurement of allergen-specific IgE in serum. 20 (10) lines of relevant allergens are coated together with 5 lines of calibrators on a carrier membrane, which is located in the well of the Polycheck[®] cassette. During incubation of the patients serum allergen-specific IgE bind to the corresponding allergens. Non-bound serum components are removed by washing. Monoclonal ligand-labeled anti-IgE-antibodies bind to allergen-bound IgE. Unbound antibodies are removed by washing. Enzyme-labeled anti-Ligands bind to the immune complexes; surplus of enzyme conjugate is removed by washing. The substrate solution is added and specifically bound enzymes convert the colourless substrate to a dark precipitate. The colour intensity of the lines is proportional to the respective allergen specific IgE concentration in the patient's serum.

With the help of Biocheck Imaging Software (BIS) and a PC the Polycheck[®] cassettes are interpreted. Each single allergen will be identified and according to the calibrator curve present in each cassette the concentration of each allergen-specific IgE is quantified.

Test Performance

Important notes:

- Place the MTP-holder onto the middle of the shaker (see picture beside).
- All test components should be at room temperature and be mixed well
- Only reagent's lots provided with the actual kit are to be used
- Powdered wash buffer should be diluted with demineralized water at least 30 minutes prior to use.
- Do not allow the membrane of the test cassettes to dry during the assay.
- All incubation steps are performed at room temperature (18-24°C) and with constant shaking.

How to put the microtitreplate onto the rocking shaker

1. Prepare a sufficient number of Polycheck[®] allergen cassettes and mark them –only on the long side of the cassette.
2. Overlay the cassettes with wash solution, then tap on absorbent paper.
3. Overlay allergen cassette with **250 µl** of Start Solution (blue cap) and incubate for **60 seconds** (always pipet into the gap). Tap carefully the cassettes on absorbent paper.
4. Add **200 µl** of the respective patients serum onto the membrane and incubate for **60 minutes** on a shaker. The orientation of the MTP-holder on the shaker should be the same during the test.
5. Decant sample and wash **three times** with **1 ml** of Polycheck[®] wash buffer. Add **250 µl** wash buffer and incubate for **5 minutes** on a shaker. Decant and wash **three times** with **1 ml** wash buffer. **Tap carefully the cassettes on absorbent paper.**
6. Pipet **250 µl** of anti-IgE Antibody (green cap) and incubate for **45 minutes** on a shaker. Decant and wash **three times** with **1 ml** wash buffer. Tap carefully the cassettes on absorbent paper.
7. Add **250 µl** Enzyme-labeled Anti-Ligand (white cap) and incubate for **20 minutes** on a shaker. Decant and wash as **described in 6**. Tap carefully the cassettes on absorbent paper.
8. Pipet **250 µl** Polycheck[®] substrate solution (black cap) and incubate for **20 minutes in the dark**. Then decant the substrate solution and wash as **described in 6**.
9. Airdry the membrane and evaluate the Polycheck[®] Allergen cassettes using a scanner and the Biocheck Imaging Software.

Interpretation of Results

For the evaluation of the Polycheck® allergen cassettes a PC and a connected flat bed scanner are used. The evaluation software Biocheck Imaging Software (BIS) analyzes and documents all patient's results. The MTP-holder with the Polycheck® allergen cassettes are placed upside down onto the flat bed scanner and read by the software. The evaluation programme checks for validity of the calibration curve, assigns the corresponding allergen to each position of the scanned band pattern and calculates allergen-specific IgE concentrations for each allergen. A patient's report is generated for each patient's sample listing the allergens tested and the present concentration of allergen specific IgE. Results are quantitatively reported in International Units (kU IgE/l). The class scores (RAST-classification) of the respective IgE concentrations and their related clinical interpretation are listed below:

IgE [kU/L]	RAST-class	Interpretation
< 0.35	0	No specific antibody detectable
0.35 – 0.7	1	Very low antibody titer; frequently no clinical evidence by sensitization
0.7 – 3.5	2	Low antibody titer; sensitization , mostly clinical evidence if in upper class ranges
3.5 – 17.5	3	Clear antibody titer; clinical evidence is mostly present
17.5 – 50	4	High antibody titer; nearly always with existing actuality
50 – 100	5	Very high antibody titer
> 100	6	Extreme high antibody titer

Limitation for the Method

1. For a final clinical diagnosis all clinical results and laboratory findings have to be regarded.
2. Allergenicity of food allergens may be affected by cooking and/or ingestion. Consequently IgE detection in *In vitro*-diagnostic assays may be diminished or even prevented despite convincing clinical history of the patient.
3. Due to occurrence of common antigenic determinants in certain allergens of a related natural material family, the possibility of cross reactions can not be excluded completely.

Assay Characteristics

Sample material:	Serum
Incubation time:	2 hours and 30 minutes at room temperature (18°-24°C) on a shaker
Lower limit of detection:	0,15 KU/L
Specificity :	human IgE; no cross reaction with IgG, IgA, IgD, IgM
Calibration:	using single donors and pool sera with defined allergen specific IgE concentrations
Method Comparison:	skin test and alternate solid phase immunoassays

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BIOCHECK GmbH